

ADHESIVE DRAPES / Plain adhesive & Iodine Adhesive Instructions for use

- Peel open the pouch and take out the drape with gloved hands.
- Peel off one side of the release liner by pulling it from edge.
- Position the drape as per orientation indicator on the liner.
- Apply the drape from one side on the operative area. Taking care to avoid entrapment of air bubbles. Simultaneously hold the other liner and keep on pulling the liner and applying the drape.
- Unfold the drape.
- ***WARNING: Some persons may be sensitive to the adhesive, if any allergic reactions appear, discontinue the use.***

PRODUCT INFORMATION DISCLOSURE

G. Surgiwear Limited has exercised reasonable care in the choice of materials and manufacture of this product. G. Surgiwear Limited excludes all warranties, whether expressed or implied by operation of law or otherwise, including, but not limited to any implied warranties of merchantability or fitness for a particular purpose. G. Surgiwear Limited shall not be liable for any incidental or consequential loss, damage or expense, directly or indirectly arising from use of this product. G. Surgiwear Limited neither assumes nor authorizes any other person to assume for it, any other or additional liability or responsibility in connection with this product.