


# CRANIOTOMY DRAPE

## D901, E911 & INT 1921

SURGIWEAR has developed this drape after feedback from surgeons. It offers many advantages: Structured construction of this drape provides:

- Level 4 protection as per ANSI/AAMI Standards.
- High performance critical area protection as per European Union standards.
- No fluid can pass through the drape in operating area.
- Drape is sterilized in a CE certified facility.
- Minimum transfer of fabric fibers into the wound.
- No chemicals present in the fabric of the drape.
- Incision drape is made of breathable lo-Drape.

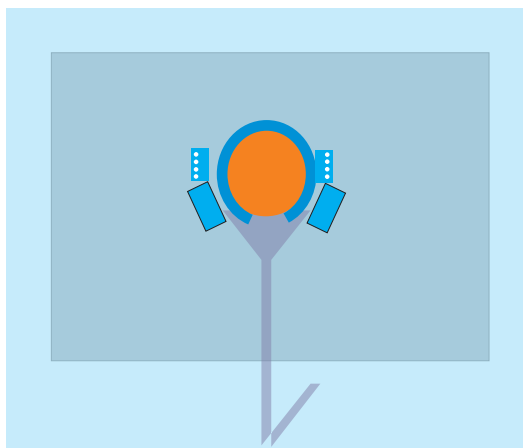

Use of this drape will result to :

- No or less post operative swelling of wound.
- Negligible post operative fever.
- Supple wound area.
- Minimum post operative pain.
- Lesser incidence of post operative infections.
- Lesser morbidity and faster recovery of patient.
- **Ultimate cost saving by way of lesser use of medicines.**

**d** Uses: For all kind of surgery on head (cranium)

### D901

**Material :** 68-77 gsm, spun lace fabric water repellent

Sheet Size 1.5 x 2.4 m

Adhesive area 20 cm dia

2 pockets 15 x 18 cm

2 Tube & Cable holders

**i** Side Channels & collecting funnel

Poly sheet lining 1.20 x 1.20 m

### INT 1921

be all free from disease & sufferings

**SURGIWEAR®**  
AN ISO 13485 : 2003 COMPANY

Adhesive area 20 cm dia  
2 pockets 15 x 18 cm  
2 Tube & Cable holders  
Side Channels & collecting funnel  
Poly sheet lining 1.20 x 1.20 mm

e

**ECO E911**

**Material :** 40-50 gsm  
SMS Polypropylene fabric water repellent

Sheet Size 1.5 x 2.4 m  
Adhesive area 20 cm dia  
Side Channels & collecting funnel  
Poly sheet lining 1.20 x 1.20 m

### Instructions for use

- Peel open the pouch and take out the drape with gloved hands.
- Peel off one side of the release liner by pulling it from edge.
- Position the drape as per orientation indicator on the liner.



operative area. Taking care to avoid entrapment of air bubbles. Simultaneously hold the other liner and keep on pulling the liner and applying the drape.

- Unfold the drape.
- **WARNING:** Some persons may be sensitive to the adhesive, if any allergic reactions appear, discontinue the use.

**SURGIWEAR®**  
AN ISO 13485 : 2003 COMPANY

European Authorised Representative  
Obelis s.a.,  
34, Av. De Tervuren, bte 44 BELGIUM  
Tel: +32 2 732 5954 Fax: +32 2 732 6003  
e-mail: mail@obelis.net

Made in India by:

**G. Surgiwear Limited**

Rasoolpur Jahanganj, Shahjahanpur-242001  
Tel: +91 5842-223818 Fax: +91 5842-222190  
e.mail: surgewear@hotmail.com

Delhi: 65454565, Mumbai: 66994258, Ahmedabad: 27545218

[www.surgewear.co.in](http://www.surgewear.co.in)