

SIMPSSERT[®] D904

Video Camera /Cable Cover



INTRODUCTION


Video cables, diathermy cables and other types of cables used in different specialties are difficult to sterilize. Moreover most of the time spare sterilized cable is not available for next surgery. To overcome this problem SURGIWEAR has developed a very simple to use and insert drape for cables and cameras.

It is made of polyethylene tube and paper leader. Distal end of PE tube has perforations and an adhesive strip for tying the end around the cable.



INSTRUCTION FOR USE

- * Press the drape from sides so that middle portion of paper tube opens up.
- * Insert the cable from above till the end comes to conical part.
- * Hold the cable at the conical end and pull it. Keep the paper tube opened.
- * Once the cable is fully inserted, leave the paper holder.
- * Tear the conical end and tie the strip around.

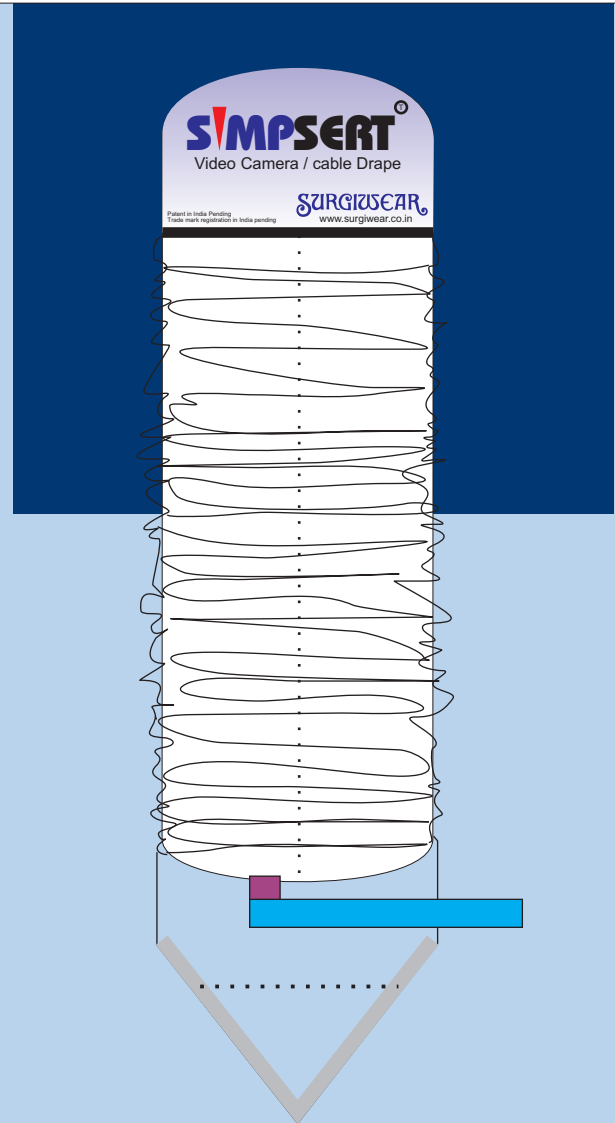

Uses: For video cable camera drape

S SIMPSSERT D904

Length 2.5 mt

Width 10 cm

Material 35 micron PE with paper leader.

Has perforations on conical end for tearing off.

Sterilized by EO

Patent pending.
Trade Mark Registration of Simsert is pending

be all free from disease & sufferings

SURGIWEAR[®]
AN ISO 13485 : 2003 COMPANY

SURGIWEAR®
AN ISO 13485 : 2003 COMPANY

www.surgiwear.co.in

European Authorised Representative
Obelis s.a.,
34, Av. De Tervuren, bte 44 BELGIUM
Tel: +32 2 732 5954 Fax: +32 2 732 6003
e-mail: mail@obelis.net

Made in India by:
G. Surgiwear Limited
Rasoolpur Jahanganj, Shahjahanpur-242001
Tel: +91 5842-223818 Fax: +91 5842-222190
e.mail: surgiwear@hotmail.com
Delhi: 65454565, Mumbai: 66994258, Ahmedabad: 27545218